

# iCARE ABO & RhD Blood Grouping Test Card

JAL [jalinnovation.com/icare-abo-rhd-blood-grouping-test-card/](http://jalinnovation.com/icare-abo-rhd-blood-grouping-test-card/)

01/03/2013

**Category Name :** Others Rapid Screen Test Kit

The assay starts with a sample applied to the sample well and added sample diluents provided. The monoclonal anti-A and anti-B immobilized respectively on the pad in sample region can react with the corresponding antigen on the surface of the red blood cells (RBC). A positive sample produces an immune response and the RBC can be captured on the sample region as a red signal, indicating that the test is positive. A negative sample does not produce an immune response, after adding sample diluents, the RBC can be washed away and the absence of the RBC indicates that the test is negative.



## Features

Short Reaction Time

Easy to Read Results

High Sensitivity 100%

No Required Aids and Equipment

Room Temperature Storage 4-30 C

Low Risk of Pollution

Shelf Life 24 Months

Low Cost

## Specifications

Category Name	Others Rapid Test Kits
Item Number	IT1064
Sample	Whole blood / Serum / Plasma
Format	Test card
Type of Test	Solid Phase
Shelf Life	24 months from date of production
Storage	2 to 30 C in sealed pouch under dry conditions

## Specifications

---

---

---

Made by [JAL Innovation \(S\) Pte Ltd](#)