



iCARE One-Step HIV 1 & 2 Rapid Test Kit (Whole Blood / Serum / Plasma)

- ① Instant ② Noticeable ③ Trustful ④ Effective ⑤ Convenient

FOR FIELD & HEALTHCARE PROFESSIONAL USE

- ▶ Sensitivity: >99%
- ▶ Specificity: >99%
- ▶ Panel Confirmation: 100%
- ▶ Detect HIV 1&2 and HIV 1 subtype "O"
- ▶ Easy to use
- ▶ Reliable result in less than 15 minutes
- ▶ Multiple independent parties evaluation including WHO, USAIDS, NICD

Test Procedures

iCare HIV 1&2 Rapid Screen Test comes sealed in an aluminum package. Do not use if the package is broken	The Alcohol Prep Pad	Tear open the Alcohol Prep Pad	Clean the finger with the Alcohol Prep Pad	Push the plunger of the Lancet and twist it twice clockwise	Remove the plunger of the Lancet
Place the Lancet at the finger and depress the trigger to prick the finger	Squeeze about 2 drops of blood for sample	Collect about 2 drops of blood samples using the Micro Pipette	Place the blood samples into the Sample Well using the Micro Pipette	Squeeze about 2 drops of Diluent Liquid into the Sample Well. Make sure the Liquid touches the Control Line	Read result within 15 minutes

Interpretation of Result

NEGATIVE RESULT (No HIV anti-bodies detected) Pink line next to "C"	POSITIVE RESULT (HIV anti-bodies detected) Pink line next to "C" and "T"	NO RESULT (The device may be faulty, please test again using a new lot)

