

iCARE Menopause (FSH) Test Kit

[JAL jalinnovation.com/icare-menopause-fsh-test-kit/](http://jalinnovation.com/icare-menopause-fsh-test-kit/)


28/02/2013

Category Name : Others Rapid Screen Test Kit

FSH (follicle-stimulating hormone) is associated with low estrogen and the stages of menopause in women. Usually, as a woman nears menopause her FSH (follicle-stimulating hormone) levels surge and notably rise. The FSH surge is generally that of ten (10) fold (approximately 110mIU/ml, can reach as high as 500mIU/ml) compared to normal levels. The FSH level could remain constant for many years after menopause. iCARE One Step FSH urine Test midstream detects this particular hormone level in urine.

When the absorbent pad is immersed into a urine specimen, capillary action carries the specimen to migrate along the membrane. When the FSH levels are equal to or above the target cutoff, there is a visible colored band in the test region which is equal to or darker than the control band. Absence of this test colored band or the test colored band weaker than the control band suggests a negative result.

To serve as a procedure control, a colored line will appear at the Control Zone region, if the test has been performed properly.

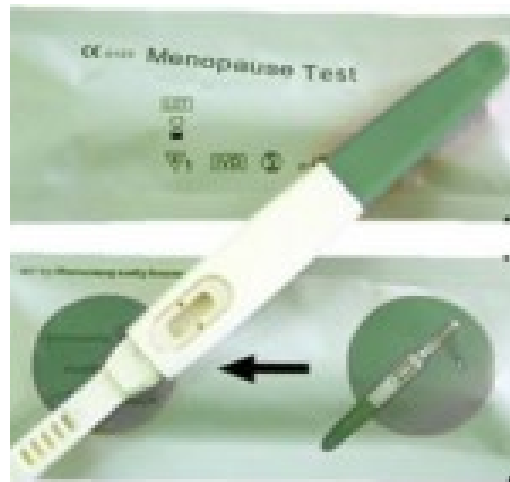

Specifications

Category Name	Others Rapid Test Kits
Item Number	WF1315 (Cassette) WF1314 (Midstream)
Sample	Urine Only
Format	Test card
Accuracy	99%
Shelf Life	24 months from date of production
Storage	2 to 30 C in sealed pouch under dry conditions
Certifications	FDA510k CE1023

Made by [JAL Innovation \(S\) Pte Ltd](#)