


## Quick Guide for iCARE Advanced Novel Coronavirus (2019-nCoV) Antigen Testing Kit (Lateral Flow Assay)

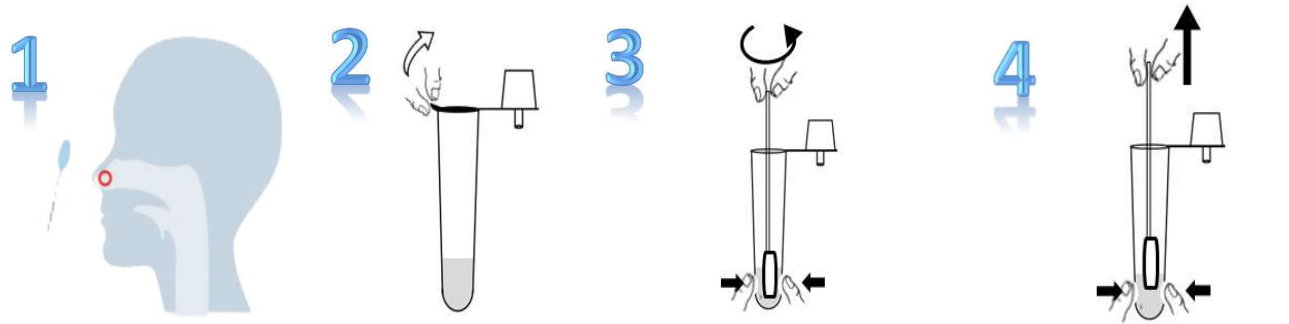


1 Collect Nasal Specimen with a swab.

2 Tear off the sealing film.

3 Put the swab specimen into the extraction tube, rotate the swab for about 15 seconds, and press the swab head against the tube wall.

4 Squeeze the swab over the head to remove the swab so as to remove as much liquid as possible from the swab.

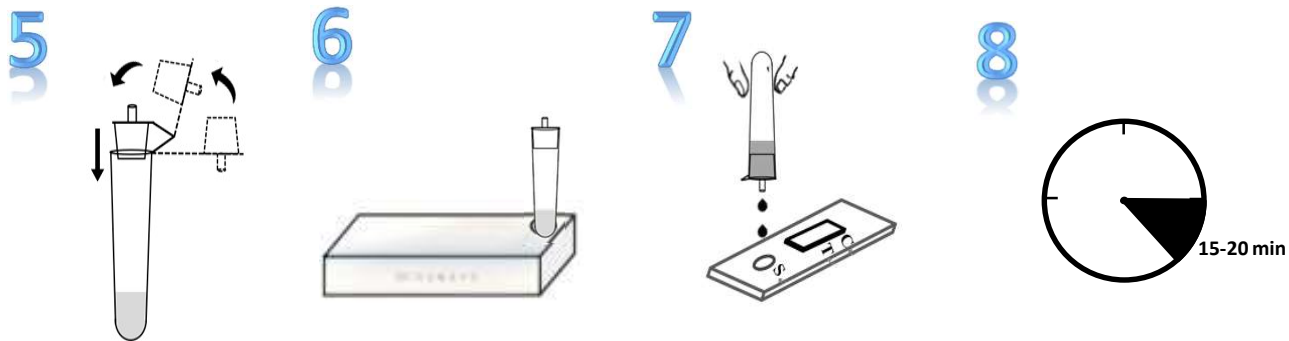


5 Install the Dripper of the tube.

6 Place the tube in the tube rack.

7 Apply 2 drops of extracted specimen to the sample well of the test Cassette, and start the timer.

8 Read the results in 15 minutes, and the result after 20 minutes is no longer valid.

### Interpretation of Test Result

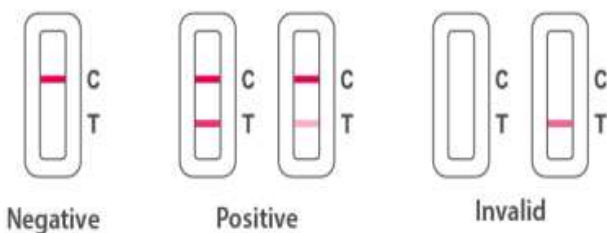

Negative

Positive

Invalid

