
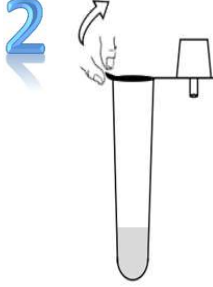

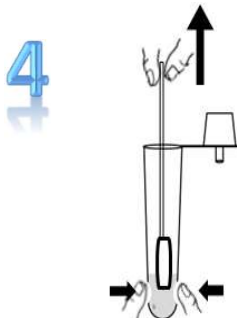


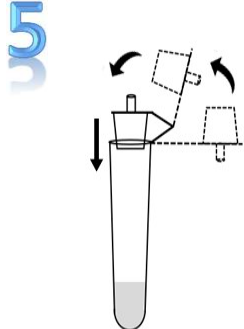
Quick Guide for iCARE Advanced Novel Coronavirus (2019-nCoV) Antigen Testing Kit (Lateral Flow Assay)

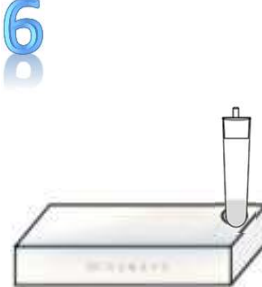
- 


1 Collect Nasal Specimen with a swab.
- 

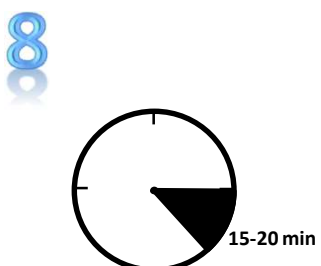
2 Tear off the sealing film.
- 

3 Put the swab specimen into the extraction tube, rotate the swab for about 15 seconds, and press the swab head against the tube wall.
- 

4 Squeeze the swab over the head to remove the swab so as to remove as much liquid as possible from the swab.
- 

5 Install the Dripper of the tube.
- 

6 Place the tube in the tube rack.
- 

7 Apply 2 drops of extracted specimen to the sample well of the test Cassette, and start the timer.
- 

8 Read the results in 15 minutes, and the result after 20 minutes is no longer valid.

Interpretation of Test Result

